


TrachFlush Quick Guide



Scan to download
user manual

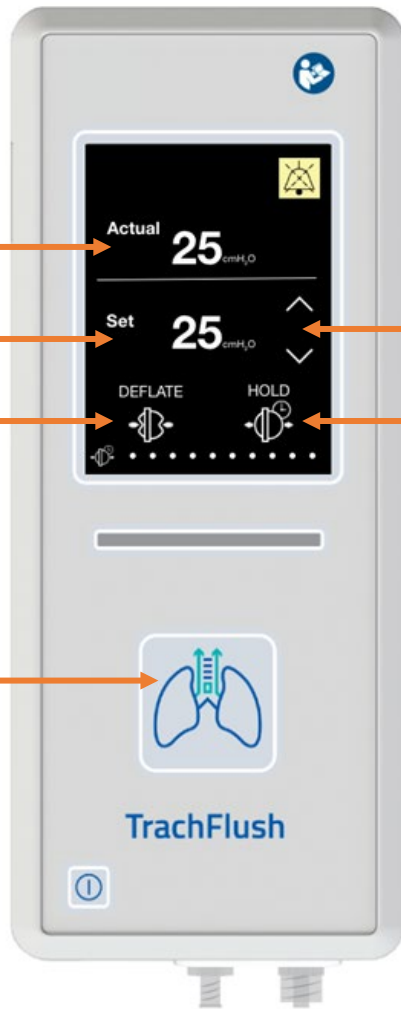

Actual Cuff Pressure

Set Cuff Pressure

Deflate

QUICK Deflate / Inflate

▲ (increase cuff pressure)

▼ (decrease cuff pressure)

Time Limited Hold

WARNINGS:






Only use on Adults

Do not use near MRI

Not for transport use

Do not clean when power is on

CONNECTIVITY AND POWER ON

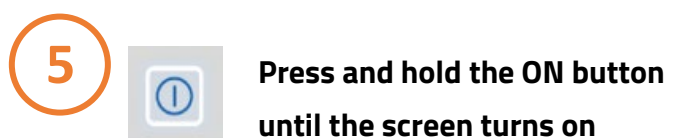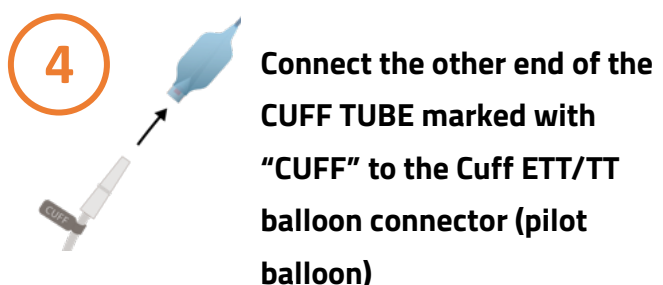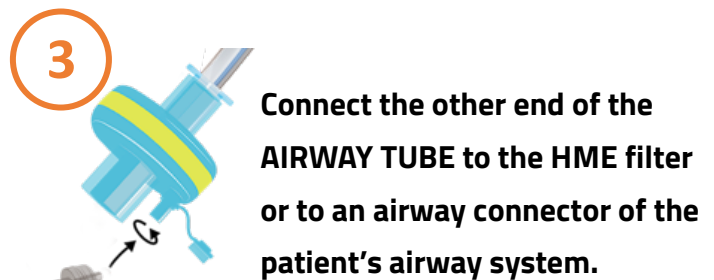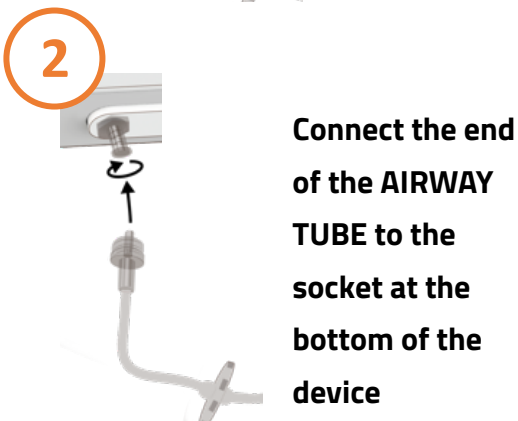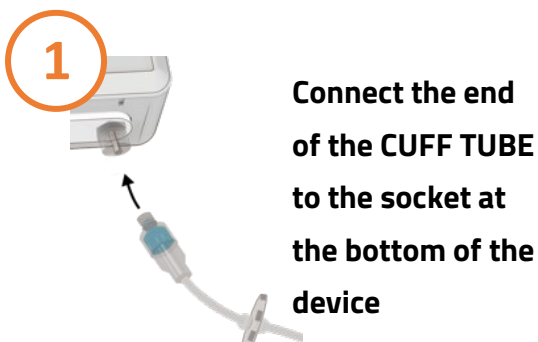


TrachFlush Quick Guide



CUFF PRESSURE

VERIFY

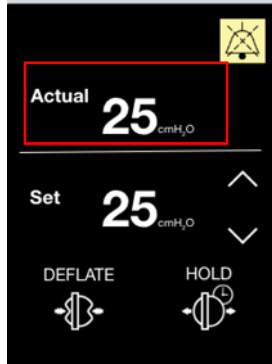

ACTUAL CUFF
PRESSURE

INCREASE



Press to INCREASE

Press to Accept


DECREASE




Press to DECREASE

Press to Accept

TIME LIMITED HOLD

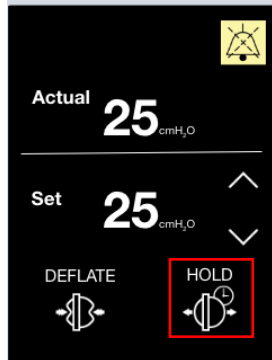

Press to activate

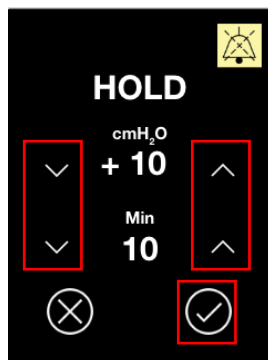

Press to change settings
and to Accept

ALARMS AND INFORMATION

SIGNALS



CUFF tube disconnected

Check Cuff connection



AIRWAY tube disconnected

Check Airway connection



CUFF leakage

Check Cuff and/or connections



CUFF above set limit


Cuff pressure too high



User shutdown denied

Disconnect tubes from TrachFlush

QUICK DEFLATE / INFLATE

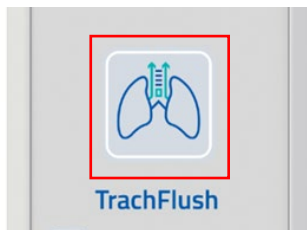

Press QUICK Deflate /
Inflate button




Press the button

DEFLATE



Press to activate

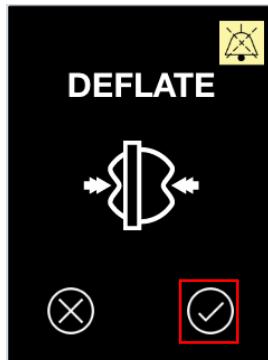

Press to Accept